




## Guide for use

### Darvall Cozy Warm Air Heater

- Mount the Cozy Heater on its four swivel wheels so air intake is elevated above the floor to minimize contamination of the HEPA air filter with dust and hair.
- Alternatively use the rear mounted clamp on an IV pole stand, or use rear hooks to hang on a kennel door.
- Connect heater to a properly grounded mains power outlet.
- The heater will not turn on unless connected to its heater hose (with built-in temperature sensor) for safety.
- Turn the rear power switch on and the heater will perform a Self-Test Sequence, then go to STANDBY mode.
- To prevent the risk of burning skin the Cozy Heater hose nozzle must be connected to a Cozy warm air blanket.
- Heater hose nozzle also push-fits into Darvall's Cozy Cwik Kennel Door Adapter used to duct warm air into kennel blankets.
- Position the blanket under or over an animal for warming - See Darvall Guide for Using Cozy Blankets
- Select the temperature you wish to operate at and press that button: 32°C (90°F); 38°C (100°F); 43°C (110°F). The LED light above the button will illuminate. Temperature selection can be changed any time.
- Press ON / OFF button to switch the Darvall Cozy Heater from Standby Mode to Operating Mode.
- Press the Air Flow Button to toggle between high and low fan speed.
- Once body temp. reaches 37°C (98.6°F), set heater to 38°C (100°F) as body temp may continue to rise 1°C/2°F once active warming ceases.
- After 500 hours of use the indicator light on the ON/OFF button will flash, the Cozy Heater will switch into standby mode and the LCD screen will display "Please replace filter". See guide in the Cozy Heater manual

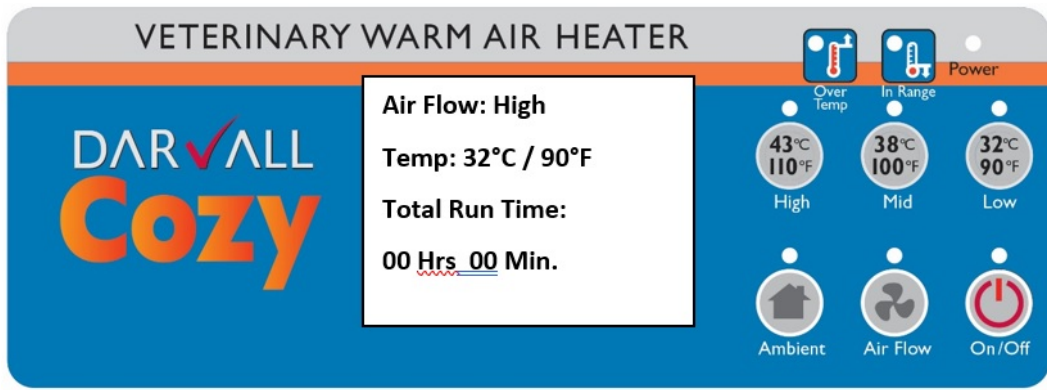

Further Information – see product manual

© Darvall