



## Guide for use

### Darvall Cozy Blankets

- The animal is positioned in contact with the white, permeable side of the blanket as indicated on the inlet port. The yellow side is impermeable to air and water.
- For surgery lay the blanket on the surgery table under the animal or position over the animal for hind-limb orthopaedic surgery.
- For dentistry place blanket underneath animal, inlet port at the tail end.
- For warming in kennels, position the blanket over the animal to minimise soiling.
- Insert the heater hose nozzle into one of the blanket inlet ports and secure with Velcro closure.
- Wrap the elasticized blanket closure tape firmly round the heater hose nozzle behind the nozzle flange, then stick the Velcro tab to the tape surface to ensure a connection that does not leak air.
- Position the animal either dorsal or lateral.
- Do not occlude inflation tube along the inlet side.
- Inflate surgery blankets after the animal is draped.
- Monitor animal's temperature frequently.
- Once body temp. reaches 37°C (98.6°F), set heater to 38°C (100°F).
- Replace blanket if hair covered, wet or soiled.

Further Information – see product manual

© Darvall